

# Unica Introkit anterior







REF 6900


- 20 pcs Unica anterior
- 20 pcs Unica minideep anterior
- 40 pcs myWedge XS
- 40 pcs myWedge S
- 40 pcs myWedge M
- 1 pc myQuickmatrix Forceps






Use only with rubber dam - For professional use only

## Refills

	Quantity	REF			
	Unica anterior	12	6912	CE	MD
	Unica anterior	50	6950	CE	MD
	Unica minideep anterior	12	6911	CE	MD
	Unica minideep anterior	50	6951	CE	MD
	myWedge, Extrasmall (XS)	100	7101	CE	MD
	myWedge, Small (S)	100	7102	CE	MD
	myWedge, Medium (M)	100	7103	CE	MD
	myQuickmatrix Forceps	1	5703	CE	MD

 Polydentia SA - via Cantonale 47  
6805 Mezzovico-Vira, Switzerland  
Made in Switzerland

phone: +41 91 946 29 48 - fax: +41 91 946 32 03  
info@polydentia.ch - polydentia.ch  
  



Follow us  
#thinkunica



more information

EC REP

IDD  
International Dental Distributors AG  
Heiligkreuz 44, 9490 Vaduz  
Liechtenstein

REV 04.08.2021

 matrix  
systems

All individual components are CE marked

Information guide

# unica anterior

developed with  
STYLE ITALIANO™




SWISS  
MADE



## matrix selection

Choose Unica according to its shape and the tooth to be restored

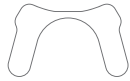

Select *Unica anterior* for:

- maxillary central incisors
- maxillary lateral incisors
- canine teeth

*New!*



Select *Unica minideep anterior* for:


- maxillary and mandibular laterals
- mandibular central incisors
- conoid teeth
- triangular teeth
- peg laterals
- teeth with narrow cervical diameter

## clinical applications



Class III cavity restorations

Clinical image: courtesy of Dr Chiodera

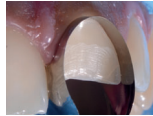

Class IV cavity restorations

Clinical image: courtesy of Dr Jorquera



Class V cavity restorations

Clinical image: courtesy of Dr Chiodera



Shape modifications

Clinical image: courtesy of Dr Shaalan



Direct stratification composite veneers

Clinical image: courtesy of Dr Manauta

## how to position Unica



1 Use the placement wings to insert Unica proximally. In case of tight contact surfaces, insert one wing at a time



2 Gently pull the matrix palatally by its placement wings



3 Push the cervical connection into the gingival sulcus



4 Insert a myWedge to fix the matrix in position

## tips and tricks

Secure Unica matrices in place with a liquid dam in case a wedge risks displacing or deforming the matrix.

